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**READ CAREFULLY!!**

The manufacturer declines any responsibility derived from the inadequate use of the product. This product has been designed and made following the directives marked for " European Directive for Sanitary Products 93/42/CEE (RD 414/1996) ", and the Regulations to which CE applies.

- The auxiliary tables are made of stainless steel AISI 304.
Maximum weight recommended for fixed shelves (distributed evenly over entire surface of the table: **25 kg**
Maximum weight recommended for adjustable shelves (distributed evenly over entire surface of the table: **15 kg**
- In case of placing weight on this one must be distributed uniformly on the whole surface of the table taking care not to damage the upholstery.
- Before moving the table, make sure there is room enough to avoid hitting other furniture or persons.
- Before moving the table, make sure all the objects on the table are safely ordered and far enough from edges to avoid accidental falls.
- In case the table is height adjustable, before working make sure the shelves are safely fixed at the desired height. If necessary, the table must be operated by 2 persons.
- Designed only for in-door use.

**MAINTENANCE**

- To avoid a premature aging of the product must follow a regular maintenance (every 15 days aprox.). Clean the Product with a wet cloth and soapy water, make sure of withdrawing the soap and drying completely with a cloth. Avoid the excesses of dampness. Avoid Aggressive cleaners like solvent, acids cleansers, bleach, ammonia, etc. In case of stain to treat IMMEDIATELY with soapy and water.
- They must be stored in dry places and in horizontal position rested on 4 legs.
- Any other procedure of conservation, it will be a responsibility of the user.

GUARANTEE

- The manufacturer guarantees the repair of the product against manufacturing defects, including labour and parts, over a period of twenty-four months (24) after its acquisition. This guarantee applies to products and accessories supplied if handled according to the use directions for use, presenting your invoice. The warranty does not apply if malfunctions are due to the normal use and / or normal wear, whether the product had been tampered with by unauthorized personnel, if damage caused by accidents, falls, beatings, exposure to degrading environments, humidity, fire, negligence, use and / or improper maintenance, etc., and generally if the cause is not attributable to the manufacturer.